


Top 4 Tips to ScripTalk Label Success!


Congratulations! Your pharmacy is going the extra mile by offering ScripTalk Talking Labels to your visually impaired patients. To make sure your patients get the most out of ScripTalk — here are the Top 4 Tips to ensure success!

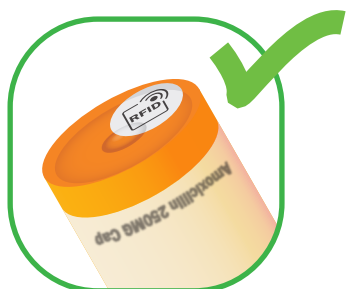

Place the label on the bottom of the medication bottle. Never place the ScripTalk Label on the container lid.



For bottles with a dimple or deep indentation, place the label slightly off-center so it lays as flat as possible. Press the RFID label firmly to ensure the adhesive sticks properly.

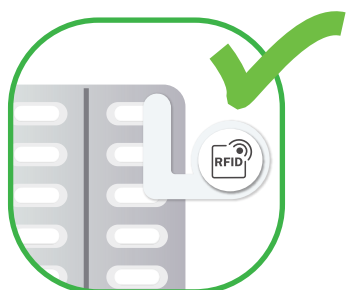

Metal can interfere with the ScripTalk Reader; use a Hang Tag for medications such as inhalers, blister packs and metal tubes.



Ensure that the RFID label is applied before giving the medication to the visually impaired patient. Never hand the label to the patient and expect them to apply it.